

Equipment for Toxopheresis

Please ensure that the following materials are available for **EACH** treatment.

At least one doctor and one nurse/medical assistant should be available for **EACH** treatment.

For preparation of the machine:

- Flush solutions:
 - 1 × 2000 ml 0.9% NaCl bags
 - 2 × 500 ml or 1000 ml 0.9% NaCl bags
- ➔ 0.9% NaCl bags with Luer lock connector **or** Spikes (included in the Toxopheresis Set)



Luer Lock connector **or** Spikes (included in the Toxopheresis Set)

For the treatment:

- Heparin sodium 25,000 IU per 5 ml
- 2 × 100 ml 5% glucose
- 2 × 500 ml 0.9% NaCl
- 2 × 100 ml 0.9% NaCl
- 2 ml and 10 ml syringes
- 1 × 200 mg magnesium (Cormagnesin 200)
- 17G dialysis needles / white cannulas
- Glucose meter
- 2 × tourniquets
- Sterile dressings
- Sterile swabs
- Skin disinfectant
- Intravenous infusion set



- Three-way stopcock with tubing (Discofix C)



- Defibrillator
- Patient monitoring device (blood pressure and ECG)
- Emergency kit DIN 13232, parts A and B
- Oxygen cylinder with masks
- ROTOCLIX (holder for tubing)

**Treatment room equipment:**

- Therapy bed / dialysis chair with armrest
- Blanket
- Pillow

Optional:

- 2 × hot water bottles or cherry pit cushions
- Bedpan and urine bottle